

Premisut POLYPROPYLENE

Description

Premisut POLYPROPYLENE suture (Polypropylene) is a sterile synthetic monofilament non-absorbable surgical suture prepared from a synthetic linear Polyolefin, Polypropylene. The molecular formula is $(C_3H_6)_n$. Premisut POLYPROPYLENE suture is coloured blue, dyed with Phthalocyanine blue.

Indications

Premisut POLYPROPYLENE suture is intended for use in general soft tissue approximation and/or ligation, including use in cardiovascular, ophthalmic and neurological tissues.

Performance

Premisut POLYPROPYLENE suture being a monofilament suture is remarkably smooth and easy to handle. It does not have interstices in its structure and being a monofilament suture, has been successfully employed in surgical wounds which could subsequently become infected or contaminated where it can minimize later sinus formation and suture extrusion. It elicits a minimal initial inflammatory reaction in tissues which is followed by gradual encapsulation of the suture by fibrous connective tissue. It is neither absorbed, nor it is subjected to degradation or weakening by the action of tissue enzymes. Due to its relative biological inertness, it is recommended for use in tissues, where the least possible suture reaction is desired. Because of its lack of adherence to tissues, it is effective as a pull out suture.

Tensile Strength

Permanent (Exceeds USP & EP requirements)

Total Absorption

None

Color

Blue



Sizes

10/0 to 1 USP

Usage Areas

General surgery, skin closure, ophthalmic, oral, gastrointestinal, gynecology, plastic, microsurgery, obstetrics, neurosurgery, orthopedic, cardiovascular, hernia repair.

Sterilization

Ethylene Oxide

Shelf Life

5 years

Packaging

Box of 12pcs / 36pcs



Premisut POLYPROPYLENE

● ½ Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x HR 6.5 PRIME NEEDLE		7/0	0.5	Blue	45	AMP731HR6.570-2	12
		7/0	0.5	Blue	75	AMP732HR6.570-2	12
		6/0	0.7	Blue	45	AMP731HR6.560-2	12
		6/0	0.7	Blue	60	AMP736HR6.560-2	12
2 X HR 8 PRIME NEEDLE		8/0	0.4	Blue	60	AMP736HR880-2	12
		7/0	0.5	Blue	60	AMP736HR870-2	12
		6/0	0.7	Blue	60	AMP736HR860-2	12
HR 10 PRIME NEEDLE		7/0	0.5	Blue	45	AMP731HR1070	12
2 x HR 10 PRIME NEEDLE		7/0	0.5	Blue	60	AMP736HR1070-2	12
		6/0	0.7	Blue	60	AMP736HR1060-2	12
		5/0	1	Blue	60	AMP736HR1050-2	36
2 x HR 13 PRIME NEEDLE		6/0	0.7	Blue	60	AMP736HR1360-2	12
		6/0	0.7	Blue	75	AMP732HR1360-2	12
		5/0	1	Blue	75	AMP732HR1350-2	36
		5/0	1	Blue	60	AMP736HR1350-2	36
		4/0	1.5	Blue	90	AMP733HR1340-2	36
2 x HR 13 VisiBlack PRIME NEEDLE		5/0	1	Blue	75	AMP732HR1350-2VB	36

Premisut POLYPROPYLENE

● ½ Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
HR 15		6/0	0.7	Blue	75	AM732HR1560	12
		5/0	1	Blue	75	AM732HR1550	36
		4/0	1.5	Blue	75	AM732HR1540	36
HR 15 PRIME NEEDLE		6/0	0.7	Blue	75	AMP732HR1560	12
		5/0	1	Blue	75	AMP732HR1550	36
		4/0	1.5	Blue	75	AMP732HR1540	36
2 x HR 16 PRIME NEEDLE		5/0	1	Blue	90	AMP733HR1650-2	36
		4/0	1.5	Blue	90	AMP733HR1640-2	36
		3/0	2	Blue	90	AMP733HR1630-2	36
2 x HR 16 VisiBlack PRIME NEEDLE		5/0	1	Blue	90	AMP733HR1650-2VB	36
		4/0	1.5	Blue	90	AMP733HR1640-2VB	36
HR 17		5/0	1	Blue	75	AM732HR1750	36
		4/0	1.5	Blue	75	AM732HR1740	36
		3/0	2	Blue	75	AM732HR1730	36

Premisut POLYPROPYLENE

● ½ Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
HR 17 PRIME NEEDLE		5/0	1	Blue	75	AMP732HR1750	36
		4/0	1.5	Blue	75	AMP732HR1740	36
		3/0	2	Blue	75	AMP732HR1730	36
HR 20 • RIBBED • RTT		4/0	1.5	Blue	75	AMP732HR2040	36
		3/0	2	Blue	75	AMP732HR2030	36
HR 20 PRIME NEEDLE • RIBBED • RTT		4/0	1.5	Blue	75	AMP732HR2040	36
		3/0	2	Blue	75	AMP732HR2030	36
2 x HR 20 • RIBBED		3/0	2	Blue	90	AMP733HR2030-2	36
		2/0	3	Blue	90	AMP733HR2020-2	36
2 x HR 20 PRIME NEEDLE • RIBBED		5/0	1	Blue	90	AMP733HR2050-2	36
		4/0	1.5	Blue	90	AMP733HR2040-2	36
		3/0	2	Blue	90	AMP733HR2030-2	36
		2/0	3	Blue	90	AMP733HR2020-2	36

Premisut POLYPROPYLENE

● ½ Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x HR 20 VisiBlack PRIME NEEDLE		5/0	1	Blue	90	AMP733HR2050-2VB	36
		4/0	1.5	Blue	90	AMP733HR2040-2VB	36
• RIBBED							
HR 22 PRIME NEEDLE		5/0	1	Blue	75	AMP732HR2250	36
• RIBBED • RTT							
HR 25		5/0	1	Blue	75	AM732HR2550	36
		4/0	1.5	Blue	75	AM732HR2540	36
		3/0	2	Blue	75	AM732HR2530	36
		2/0	3	Blue	75	AM732HR2520	36
		0	3.5	Blue	75	AM732HR2510	36
		1	4	Blue	75	AM732HR2501	36
• RIBBED • RTT							
HR 25 PRIME NEEDLE		5/0	1	Blue	75	AMP732HR2550	36
		4/0	1.5	Blue	75	AMP732HR2540	36
		3/0	2	Blue	75	AMP732HR2530	36
		2/0	3	Blue	75	AMP732HR2520	36
		0	3.5	Blue	75	AMP732HR2510	36
		1	4	Blue	75	AMP732HR2501	36
• RIBBED • RTT							
2 x HR 25		4/0	1.5	Blue	90	AM733HR2540-2	36
		3/0	2	Blue	90	AM733HR2530-2	36
		2/0	3	Blue	90	AM733HR2520-2	36
• RIBBED							

Premisut POLYPROPYLENE

● ½ Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x HR 25 PRIME NEEDLE		4/0	1.5	Blue	90	AMP733HR2540-2	36
		3/0	2	Blue	90	AMP733HR2530-2	36
		2/0	3	Blue	90	AMP733HR2520-2	36
• RIBBED							
2 x HR 25 VisiBlack PRIME NEEDLE		5/0	1	Blue	90	AMP733HR2550-2VB	36
		4/0	1.5	Blue	90	AMP733HR2540-2VB	36
		3/0	2	Blue	90	AMP733HR2530-2VB	36
		2/0	3	Blue	90	AMP733HR2520-2VB	36
• RIBBED							
HR 30		3/0	2	Blue	75	AM732HR3030	36
		2/0	3	Blue	75	AM732HR3020	36
		0	3.5	Blue	75	AM732HR3010	36
		1	4	Blue	75	AM732HR3001	36
• RIBBED • RTT							
HR 30 PRIME NEEDLE		3/0	2	Blue	75	AMP732HR3030	36
		2/0	3	Blue	75	AMP732HR3020	36
		0	3.5	Blue	75	AMP732HR3010	36
• RIBBED • RTT							
HR 30 Heavy PRIME NEEDLE		1	4	Blue	75	AMP732HR3001H	36
• RIBBED • RTT • HEAVY NEEDLE							

Premisut POLYPROPYLENE

● ½ Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x HR 30		3/0	2	Blue	90	AM733HR3030-2	36
		2/0	3	Blue	90	AM733HR3020-2	36
• RIBBED							
2 x HR 30 PRIME NEEDLE		3/0	2	Blue	90	AMP733HR3030-2	36
		2/0	3	Blue	90	AMP733HR3020-2	36
• RIBBED							
HR 35		3/0	2	Blue	75	AM732HR3530	36
		3/0	2	Blue	90	AM733HR3530	36
		2/0	3	Blue	75	AM732HR3520	36
		0	3.5	Blue	75	AM732HR3510	36
		0	3.5	Blue	90	AM733HR3510	36
		1	4	Blue	75	AM732HR3501	36
• RIBBED • RTT							
HR 35 PRIME NEEDLE		3/0	2	Blue	75	AMP732HR3530	36
		3/0	2	Blue	90	AMP733HR3530	36
		2/0	3	Blue	75	AMP732HR3520	36
		0	3.5	Blue	75	AMP732HR3510	36
		0	3.5	Blue	90	AMP733HR3510	36
		1	4	Blue	75	AMP732HR3501	36
• RIBBED • RTT							
HR 40		0	3.5	Blue	75	AM732HR4010	36
		1	4	Blue	75	AM732HR4001	36
		2	5	Blue	75	AM732HR4002	36
• RIBBED • RTT							

Premisut POLYPROPYLENE

● ½ Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
HR 40 PRIME NEEDLE		0	3.5	Blue	75	AMP732HR4010	36
		1	4	Blue	75	AMP732HR4001	36
		2	5	Blue	75	AMP732HR4002	36
2 x HR 48		0	3.5	Blue	90	AM733HR4810-2	36
		1	4	Blue	90	AM733HR4801-2	36
2 x HR 48 PRIME NEEDLE		0	3.5	Blue	90	AMP733HR4810-2	36
		1	4	Blue	90	AMP733HR4801-2	36
HR 48		0	3.5	Blue	90	AM733HR4810	36
HR 48 PRIME NEEDLE		0	3.5	Blue	90	AMP733HR4810	36

Premisut POLYPROPYLENE

 1/2 circle round bodied TAPERCUT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x HRT 17 PRIME NEEDLE		5/0	1	Blue	90	AMP733HRT1750-2	36
		4/0	1.5	Blue	90	AMP733HRT1740-2	36
		3/0	2	Blue	90	AMP733HRT1730-2	36
		2/0	3	Blue	90	AMP733HRT1720-2	36
HRT 17		5/0	1	Blue	75	AM732HRT1750	36
		4/0	1.5	Blue	75	AM732HRT1740	36
		3/0	2	Blue	75	AM732HRT1730	36
HRT 17 PRIME NEEDLE		5/0	1	Blue	75	AMP732HRT1750	36
		4/0	1.5	Blue	75	AMP732HRT1740	36
		3/0	2	Blue	75	AMP732HRT1730	36
2 x HRT 25		2/0	3	Blue	90	AM733HRT2520-2	36
		0	3.5	Blue	90	AMP733HRT2510-2	36
• RIBBED							
2 x HRT 25 PRIME NEEDLE		4/0	1.5	Blue	90	AMP733HRT2540-2	36
		3/0	2	Blue	90	AMP733HRT2530-2	36
		2/0	3	Blue	90	AMP733HRT2520-2	36
		0	3.5	Blue	90	AMP733HRT2510-2	36
• RIBBED							

Premisut POLYPROPYLENE

 **1/2 circle round bodied TAPERCUT** 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
HRT 25		4/0	1.5	Blue	75	AM732HRT2540	36
		3/0	2	Blue	75	AM732HRT2530	36
		2/0	3	Blue	75	AM732HRT2520	36
HRT 25 PRIME NEEDLE		4/0	1.5	Blue	75	AMP732HRT2540	36
		3/0	2	Blue	75	AMP732HRT2530	36
		2/0	3	Blue	75	AMP732HRT2520	36
2 x HRT 30		3/0	2	Blue	90	AM733HRT3030-2	36
		2/0	3	Blue	90	AM733HRT3020-2	36
2 x HRT 30 PRIME NEEDLE		3/0	2	Blue	90	AMP733HRT3030-2	36
		2/0	3	Blue	90	AMP733HRT3020-2	36

 1/2 circle round bodied TAPERCUT

1/2 circle Reverse Cutting

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
HS 17 PRIME NEEDLE		3/0	2	Blue	45	AMP731HS1730	36
HS 20		3/0	2	Blue	75	AM732HS2030	36
• RIBBED • RTT							

Premisut POLYPROPYLENE

▼ 1/2 circle Reverse Cutting

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
HS 40 PRIME NEEDLE • RIBBED • RTT		0	1.5	Blue	75	AMP732HS4010	36

▲ 1/2 Circle Cutting

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
HNS 20 • RTT		4/0	1.5	Blue	75	AM732HNS2040	36
HNS 24 • RTT		3/0	2	Blue	75	AM732HNS2430	36
HNS 26 • RTT		2/0	3	Blue	75	AM732HNS2620	36

Premisut POLYPROPYLENE

▲ 1/2 Circle Cutting

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
HNS 40		0	1.5	Blue	75	AM732HNS4010	36
• RTT							

▼ 1/2 Circle Extra Reverse Cutting – Long cutting edge (plastic/cosmetic)

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
XHS 16 PRIME NEEDLE		4/0	1.5	Blue	45	AMP731XHS1640	36

⊙ 1/2 circle round bodied Soft Cutting Coronary – Calcified Coronary

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x HCC 13 PRIME NEEDLE		6/0	0.7	Blue	60 long pack	AMP736HCC1360-2LP	12
		5/0	1	Blue	75 long pack	AMP732HCC1350-2LP	12
		5/0	1	Blue	60 long pack	AMP736HCC1350-2LP	12
		4/0	1.5	Blue	75	AMP732HCC1340-2	12
2 x HCC 16 PRIME NEEDLE		3/0	2	Blue	90	AMP733HCC1630-2	12

Premisut POLYPROPYLENE

1/2 circle round bodied Soft Cutting Coronary – Calcified Coronary

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x HCC 16 VisiBlack PRIME NEEDLE		6/0	0.7	Blue	90 long pack	AMP733HCC1660-2VBLP	12
		5/0	1	Blue	90 long pack	AMP733HCC1650-2VBLP	12
2 x HCC 31 PRIME NEEDLE		3/0	2	Blue	90	AMP733HCC3130-2	12

3/8 Circle Reverse Cutting

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
DS 10 PRIME NEEDLE		6/0	0.7	Blue	45	AMP731DS1060	12
DS 12 PRIME NEEDLE		7/0	0.5	Blue	45	AMP731DS1270	12
		6/0	0.7	Blue	45	AMP731DS1260	12
		5/0	1	Blue	45	AMP731DS1250	36
		4/0	1.5	Blue	75	AMP732DS1240	36
DS 16		6/0	0.7	Blue	45	AM731DS1660	12
		6/0	0.7	Blue	75	AM732DS1660	12
		5/0	1	Blue	45	AM731DS1650	36
		4/0	1.5	Blue	75	AM732DS1640	36
		3/0	2	Blue	75	AM732DS1630	36

Premisut POLYPROPYLENE

3/8 Circle Reverse Cutting

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
DS 16 PRIME NEEDLE		6/0	0.7	Blue	45	AMP731DS1660	12
		6/0	0.7	Blue	75	AMP732DS1660	12
		5/0	1	Blue	45	AMP731DS1650	36
		4/0	1.5	Blue	75	AMP732DS1640	36
		3/0	2	Blue	75	AMP732DS1630	36
DS 19 • RIBBED • RTT *NO RTT		6/0	0.7	Blue	45	AM731DS1960*	12
		6/0	0.7	Blue	75	AM732DS1960*	12
		5/0	1	Blue	45	AM731DS1950	36
		5/0	1	Blue	75	AM732DS1950	36
		4/0	1.5	Blue	45	AM731DS1940	36
		4/0	1.5	Blue	75	AM732DS1940	36
		3/0	2	Blue	45	AM731DS1930	36
		3/0	2	Blue	75	AM732DS1930	36
		2/0	3	Blue	75	AM732DS1920	36
DS 19 PRIME NEEDLE • RIBBED • RTT *NO RTT		6/0	0.7	Blue	45	AMP731DS1960*	12
		6/0	0.7	Blue	75	AMP732DS1960*	12
		5/0	1	Blue	45	AMP731DS1950	36
		4/0	1.5	Blue	45	AMP731DS1940	36
		4/0	1.5	Blue	75	AMP732DS1940	36
		3/0	2	Blue	45	AMP731DS1930	36
		3/0	2	Blue	75	AMP732DS1930	36
		2/0	3	Blue	75	AMP732DS1920	36
DS 24 • RIBBED • RTT		4/0	1.5	Blue	75	AM732DS2440	36
		3/0	2	Blue	75	AM732DS2430	36
		2/0	3	Blue	75	AM732DS2420	36
		0	3.5	Blue	75	AM732DS2410	36
		1	4	Blue	75	AM732DS2401	36

Premisut POLYPROPYLENE

▼ 3/8 Circle Reverse Cutting

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
DS 24 PRIME NEEDLE		4/0	1.5	Blue	75	AMP732DS2440	36
		3/0	2	Blue	75	AMP732DS2430	36
		2/0	3	Blue	75	AMP732DS2420	36
		0	3.5	Blue	75	AMP732DS2410	36
		1	4	Blue	75	AMP732DS2401	36
DS 26		2/0	3	Blue	75	AM732DS2620	36
		0	3.5	Blue	75	AM732DS2610	36
DS 26 PRIME NEEDLE		2/0	3	Blue	75	AMP732DS2620	36
		0	3.5	Blue	75	AMP732DS2610	36
DS 30		4/0	1.5	Blue	75	AM732DS3040	36
		3/0	2	Blue	75	AM732DS3030	36
		2/0	3	Blue	75	AM732DS3020	36
		0	3.5	Blue	75	AM732DS3010	36
		1	4	Blue	75	AM732DS3001	36
DS 30 PRIME NEEDLE		4/0	1.5	Blue	75	AMP732DS3040	36
		3/0	2	Blue	75	AMP732DS3030	36
		2/0	3	Blue	75	AMP732DS3020	36
		0	3.5	Blue	75	AMP732DS3010	36
		1	4	Blue	75	AMP732DS3001	36

- RIBBED
- RTT

- RIBBED
- RTT

- RIBBED
- RTT

- RIBBED
- RTT

- RIBBED
- RTT

Premisut POLYPROPYLENE

▼ 3/8 Circle Reverse Cutting 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
DS 36 							

Premisut POLYPROPYLENE

▼ 3/8 Circle Reverse Cutting 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
DS 45		2/0	3	Blue	90	AM733DS4520	36
		0	3.5	Blue	90	AM733DS4510	36
		1	4	Blue	90	AM733DS4501	36
• RIBBED							
DS 45 PRIME NEEDLE		2/0	3	Blue	90	AMP733DS4520	36
		0	3.5	Blue	90	AMP733DS4510	36
		1	4	Blue	90	AMP733DS4501	36
• RIBBED							
DS 60		2/0	3	Blue	90	AM733DS6020	36
		0	3.5	Blue	90	AM733DS6010	36
• RIBBED							
DS 60 PRIME NEEDLE		2/0	3	Blue	90	AMP733DS6020	36
		0	3.5	Blue	90	AMP733DS6010	36
• RIBBED							

Premisut POLYPROPYLENE

▼ 3/8 Circle Reverse Cutting – Long cutting edge (Extra Cutting) 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
XDS 8 PRIME NEEDLE		7/0	0.5	Blue	45	AMP731XDS870	12
		6/0	0.7	Blue	45	AMP731XDS860	12
XDS 11 PRIME NEEDLE		7/0	0.5	Blue	75	AMP732XDS1170	12
		6/0	0.7	Blue	45	AMP731XDS1160	12
		6/0	0.7	Blue	75	AMP732XDS1160	12
XDS 13 PRIME NEEDLE		6/0	0.7	Blue	45	AMP731XDS1360	12
		6/0	0.7	Blue	75	AMP732XDS1360	12
		5/0	1	Blue	45	AMP731XDS1350	36
		5/0	1	Blue	75	AMP732XDS1350	36
		4/0	1.5	Blue	45	AMP731XDS1340	36
		4/0	1.5	Blue	75	AMP732XDS1340	36
XDS 16 PRIME NEEDLE		6/0	0.7	Blue	75	AMP732XDS1660*	12
		5/0	1	Blue	45	AMP731XDS1650	36
		5/0	1	Blue	75	AMP732XDS1650	36
		4/0	1.5	Blue	45	AMP731XDS1640	36
		4/0	1.5	Blue	75	AMP732XDS1640	36
XDS 19 PRIME NEEDLE		5/0	1	Blue	45	AMP731XDS1950	36
		5/0	1	Blue	75	AMP732XDS1950	36
		4/0	1.5	Blue	45	AMP731XDS1940	36
		4/0	1.5	Blue	75	AMP732XDS1940	36
		3/0	2	Blue	75	AMP732XDS1930	36
• RTT							

Premisut POLYPROPYLENE

3/8 Circle Reverse Cutting – Long cutting edge (Extra Cutting)

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
XDS 24 PRIME NEEDLE		4/0	1.5	Blue	45	AMP731XDS2440	36
		4/0	1.5	Blue	75	AMP732XDS2440	36
		3/0	2	Blue	60	AMP736XDS2430	36
		3/0	2	Blue	75	AMP732XDS2430	36
• RTT							
XDS 26 PRIME NEEDLE		2/0	3	Blue	75	AMP732XDS2620	36
• RTT							
XDS 30 PRIME NEEDLE		2/0	3	Blue	75	AMP732XDS3020	36
• RTT							

3/8 Circle Conventional Extra Cutting – Long cutting edge (Extra Cutting/cosmetic)

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
XNS 16 PRIME NEEDLE		6/0	0.7	Blue	45	AMP731XNS1660	12
		6/0	0.7	Blue	75	AMP732XNS1660	12
		5/0	1	Blue	45	AMP731XNS1650	36
		5/0	1	Blue	75	AMP732XNS1650	36
		4/0	1.5	Blue	45	AMP731XNS1640	36
		4/0	1.5	Blue	75	AMP732XNS1640	36

Premisut POLYPROPYLENE

3/8 Circle Conventional Extra Cutting – Long cutting edge (Extra Cutting/cosmetic)

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
XNS 19 PRIME NEEDLE		5/0	1	Blue	45	AMP731XNS1950	36
		5/0	1	Blue	75	AMP732XNS1950	36
		4/0	1.5	Blue	45	AMP731XNS1940	36
		4/0	1.5	Blue	75	AMP732XNS1940	36
• RTT							
XNS 26 PRIME NEEDLE		4/0	1.5	Blue	45	AMP731XNS2640	36
		3/0	2	Blue	45	AMP731XNS2630	36
• RTT							

3/8 Circle Round bodied TAPERPOINT

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
DR 3.8 PRIME NEEDLE		10/0	0.2	Blue	15	AMP739DR3.8100	12
DR 4.8 PRIME NEEDLE		10/0	0.2	Blue	15	AMP739DR4.8100	12
		9/0	0.3	Blue	15	AMP739DR4.890	12
		8/0	0.4	Blue	15	AMP739DR4.880	12
DR 5.1 PRIME NEEDLE		10/0	0.2	Blue	15	AMP739DR5.1100	12
		9/0	0.3	Blue	15	AMP739DR5.190	12

Premisut POLYPROPYLENE

● 3/8 Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x DR 5.1 PRIME NEEDLE		9/0	0.3	Blue	15	AMP739DR5.190-2	12
DR 6.5 PRIME NEEDLE		9/0	0.3	Blue	15	AMP739DR6.590	12
2 x DR 8 PRIME NEEDLE		8/0	0.4	Blue	60	AMP736DR880-2	12
		7/0	0.5	Blue	60	AMP736DR870-2	12
2 x DR 9 VisiBlack PRIME NEEDLE		7/0	0.5	Blue	60	AMP736DR970-2VB	12
		6/0	0.7	Blue	60	AMP736DR960-2VB	12
2 x DR 9.3 PRIME NEEDLE		6/0	0.7	Blue	75	AMP732DR9.360-2	12
DR 9.3 PRIME NEEDLE		6/0	0.7	Blue	75	AMP732DR9.360	12

Premisut POLYPROPYLENE

● 3/8 Circle Round bodied TAPERPOINT 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x DR 13 PRIME NEEDLE		7/0	0.5	Blue	60	AMP736DR1370-2	12
		7/0	0.5	Blue	75	AMP732DR1370-2	12
		6/0	0.7	Blue	60	AMP736DR1360-2	12
		6/0	0.7	Blue	75	AMP732DR1360-2	12
		5/0	1	Blue	45	AMP731DR1350-2	36
		5/0	1	Blue	60	AMP736DR1350-2	36
		5/0	1	Blue	75	AMP732DR1350-2	36
		5/0	1	Blue	90	AMP733DR1350-2	36
		4/0	1.5	Blue	90	AMP733DR1340-2	36
DR 13 PRIME NEEDLE		6/0	0.7	Blue	75	AMP732DR1360	12
		5/0	1	Blue	75	AMP732DR1350	36
		4/0	1.5	Blue	75	AMP732DR1340	36
2 x DR 17 PRIME NEEDLE		5/0	1	Blue	90	AMP733DR1750-2	36
		4/0	1.5	Blue	90	AMP733DR1740-2	36
DR 17 PRIME NEEDLE		5/0	1	Blue	75	AMP732DR1750	36
		4/0	1.5	Blue	75	AMP732DR1740	36

Premisut POLYPROPYLENE

 **3/8 circle round bodied TAPERCUT** 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x DRT 5.5 PRIME NEEDLE		8/0	0.4	Blue	60	AMP736DRT5.580-2	12
2 x DRT 13 PRIME NEEDLE		6/0	0.7	Blue	75	AMP732DRT1360-2	12
		5/0	1	Blue	75	AMP732DRT1350-2	36
DRT 13 PRIME NEEDLE		6/0	0.7	Blue	75	AMP732DRT1360	12
		5/0	1	Blue	75	AMP732DRT1350	36
2 x DRT 17 PRIME NEEDLE		5/0	1	Blue	75	AMP732DRT1750-2	36
		4/0	1.5	Blue	75	AMP732DRT1740-2	36
		4/0	1.5	Blue	90	AMP733DRT1740-2	36
DRT 17 PRIME NEEDLE		5/0	1	Blue	75	AMP732DRT1750	36
		4/0	1.5	Blue	75	AMP732DRT1740	36
		3/0	2	Blue	75	AMP732DRT1730	36

Premisut POLYPROPYLENE

3/8 circle round bodied TAPERCUT

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x DRT 26 PRIME NEEDLE		3/0	2	Blue	90	AMP733DRT2630-2	36

3/8 circle squared bodied Soft Cutting Coronary – Calcified Coronary

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x DCC 8 VisiBlack PRIME NEEDLE		8/0	0.4	Blue	60 long pack	AMP736DCC880-2VBLP	12
2 x DCC 9.3 PRIME NEEDLE		8/0	0.4	Blue	60 long pack	AMP736DCC9.380-2LP	12
		7/0	0.5	Blue	45 long pack	AMP731DCC9.370-2LP	12
		7/0	0.5	Blue	75 long pack	AMP732DCC9.370-2LP	12
		6/0	0.7	Blue	60 long pack	AMP736DCC9.360-2LP	12
		5/0	1	Blue	60 long pack	AMP736DCC9.350-2LP	12
DCC 9.3 PRIME NEEDLE		7/0	0.5	Blue	75 long pack	AMP732DCC9.370LP	12
2 x DCC 9.3 VisiBlack PRIME NEEDLE		7/0	0.5	Blue	75 long pack	AMP732DCC9.370-2VBLP	12
		6/0	0.7	Blue	75 long pack	AMP732DCC9.360-2VBLP	12

Premisut POLYPROPYLENE



3/8 circle squared bodied Soft Cutting Coronary – Calcified Coronary



NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x DCC 11 PRIME NEEDLE		6/0	0.7	Blue	60 long pack	AMP736DCC1160-2LP	12
2 x DCC 11 VisiBlack PRIME NEEDLE		6/0	0.7	Blue	60 long pack	AMP736DCC1160-2VBLP	12
2 x DCC 13 PRIME NEEDLE		6/0	0.7	Blue	45 long pack	AMP731DCC1360-2LP	12
		6/0	0.7	Blue	75 long pack	AMP732DCC1360-2LP	12
		6/0	0.7	Blue	2 x 75 long pack	AMP732D-CC1360.2-2LP	12
		5/0	1	Blue	75 long pack	AMP732DCC1350-2LP	12
		5/0	1	Blue	90 long pack	AMP733DCC1350-2LP	12
DCC 13 PRIME NEEDLE		6/0	0.7	Blue	75 long pack	AMP732DCC1360LP	12
		5/0	1	Blue	75 long pack	AMP732DCC1350LP	12

Premisut POLYPROPYLENE

● Compound curve TAPERPOINT

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x PR 9.3 PRIME NEEDLE		7/0	0.5	Blue	75	AMP732PR9.370-2	12
2 x PR 13 PRIME NEEDLE		6/0	0.7	Blue	75	AMP732PR1360-2	12
		5/0	1	Blue	75	AMP732PR1350-2	36

● 5/8 Circle Round bodied TAPERPOINT

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
FR 26 PRIME NEEDLE		2/0	3	Blue	75	AMP732FR2620	36
• RIBBED • RTT							

● Straight needle Round bodied TAPERPOINT

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x GR 45 PRIME NEEDLE		3/0	2	Blue	45	AMP731GR4530-2	36
• RIBBED							

Premisut POLYPROPYLENE

▲ Straight Cutting

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
GS 60		3/0	2	Blue	75	AM732GS6030	36
		2/0	3	Blue	75	AM732GS6020	36
• RIBBED							
GS 60 PRIME NEEDLE		3/0	2	Blue	75	AMP732GS6030	36
		2/0	3	Blue	75	AMP732GS6020	36
• RIBBED							

▼ 3/8 Circle MICROPOINT SPATULA

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
DSP 5.5 PRIME NEEDLE		10/0	0.2	Blue	30	AMP734DSP5.5100	12
2 x DSP 5.5 PRIME NEEDLE		10/0	0.2	Blue	30	AMP734DSP5.5100-2	12
2 X DSP 6 PRIME NEEDLE		10/0	0.2	Blue	30	AMP734DSP6100-2	12

Premisut POLYPROPYLENE


 Straight MICROPOINT SPATULA
 

NEEDLE LENGTH MM		USP	EP	COLOUR	THREAD LENGTH CM	ATLAS REF	PCS/BOX
2 x GSP 16 PRIME NEEDLE		10/0	0.2	Blue	30	AMP734GSP16100-2	12
GSP 16 PRIME NEEDLE		10/0	0.2	Blue	15 loop	AMP739GSP16L100	12